


POSITIVE PRESSURE

Flush Technique for the CLAVE[®] Connector

It is recommended to positive pressure flush the CLAVE Connector to prevent the backflow of blood into the catheter upon disconnection.

DIRECTIONS FOR FLUSHING:

1. Flush CLAVE Connector as normally prescribed after each use per facility protocol. Use about 1cc of the flush solution to conduct the positive pressure flush.
2. Without disconnecting flush syringe, grasp syringe barrel firmly with thumb and fingers and position index finger on top of syringe plunger.
3. While disconnecting syringe allow index finger to rest firmly on plunger in order to push the remaining 1cc of flush from syringe.
4. This procedure will allow you to completely fill the catheter, and prevent the backflow of blood into the catheter lumen upon disconnection.

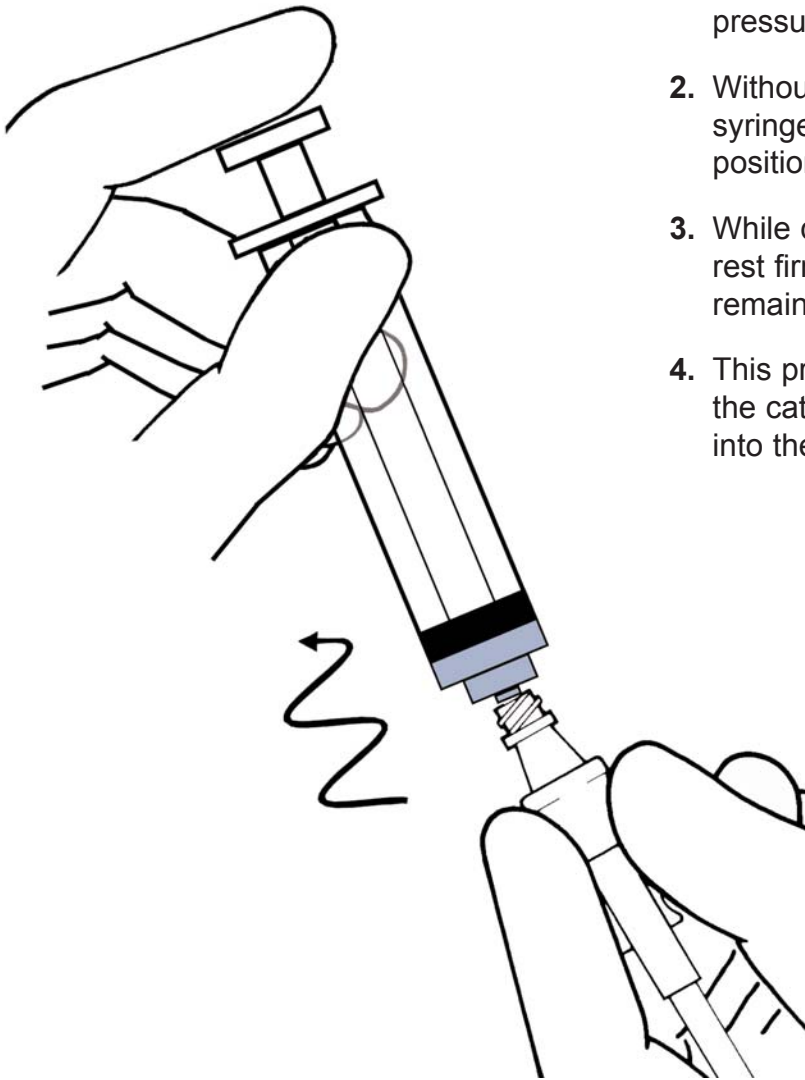

Cautions:

- Do Not Use Needles.
- Flush CLAVE after each use per facility protocol.
- Federal (USA) law restricts this device to sale by or on the order of a physician.
- Change device per (CDC) guidelines or per validated facility protocol.